

Getting started with TraumaCad


Enhance your pre-op planning experience

TraumaCad is a pre-operative planning software allowing orthopedic surgeons to plan surgeries, template patient images, take measurements, and simulate the expected result prior to the surgery.

For over a decade, TraumaCad has set the standard in orthopedic digital templating.



We welcome you to enjoy the benefits of our experience.

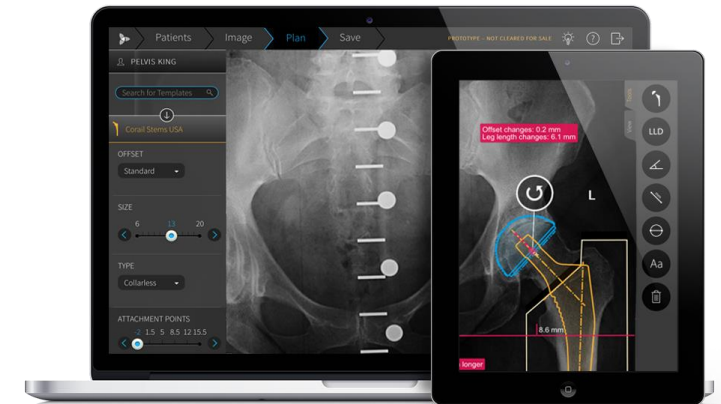

Before registration

TraumaCad Web and Mobile are part of Qentry, the Brainlab Cloud service for healthcare professionals. Qentry is a HIPAA and GDPR compliant platform for a number of medical web applications. Qentry also provides fast access and data management for medical images to thousands of users around the world.

All TraumaCad users need to [register](#) independently to the service in order to generate credentials and access TraumaCad Web and Mobile- the registration is free of charge.

Registration instructions

1. Visit quentry.com to register for a free Quentry account.
Note: Make sure to fill in the Orthopedics option under the specialty field.
2. Once registered, an activation link is sent to your email address.
Hint: The email may be hiding in your Junk folder.
3. Complete your activation by filling the activation form.
Note: Make sure to tick “Request TraumaCad demo license”.
4. Our team will contact you and assign a trial license to you.
(Requests are being validated within 2-3 business days.)

Essential information



Read the TraumaCad 2.5 [Quick Start Guide](#)



Watch video tutorials on our [YouTube channel](#)



Demo images can be found on your Qentry account


Visit us online!

[User Group](#) | [Facebook](#) | [YouTube](#) | [Instagram](#) | [traumacad.com](#)

Install TraumaCad Web

After a license was provided to you, you can easily download TraumaCad Web.

1. Login to your Qentry account
2. Go to Applications (appears in the left menu)
3. Click on the install button in the TraumaCad Web section



TRAUMACAD® WEB

TraumaCad is a pre-operative planning software allowing orthopedic surgeons to plan surgeries, template patient images, take measurements and simulate the expected result prior to the surgery. Fully integrated with Qentry, the software includes a large digital template library and offers a full set of measurement tools for orthopedic departments, including THR, TKR, Trauma, Pediatric, Deformity correction, Foot & Ankle, Spine and Upper Limb. [Read More](#)

Install

Login to TraumaCad Mobile

After a license was provided to you, you can access TraumaCad Mobile online.

1. Go to: <https://www.quentry.com/traumacadmobile/#/Login>
2. Login using your credentials



To access TraumaCad Mobile on an iPad, download the app from the App Store.

Frequently asked questions

Q I tried to sign up, but I couldn't find my country on the list.

A As a medical device company, Brainlab is committed to complying with local regulations. TraumaCad may not be cleared for sale in your country.

Q I didn't receive an activation link.

A Make sure to choose orthopedics specialty. If that still doesn't work, please contact our [support team](#) to re-send it.

Q I've completed registration but haven't received the demo license yet.

A Your request is being processed- it takes 2-3 business days.

Q I need more demo images.

A Please [contact us](#).